Intended Use

The DreamStation Cellular Modem is designed for use with select Philips Respironics therapy devices. Once installed, it automatically transfers data between the therapy device and Philips Respironics proprietary compliance software. The DreamStation Cellular Modem is able to receive oximetry data and transfer it to the therapy device.

Package Contents

DreamStation Cellular Modem

Warnings and Cautions

Warnings

- If you notice any unexplained changes in the performance of this device, if the device is dropped or mishandled, if water is spilled into the enclosure, or if the enclosure is broken, discontinue use and contact Philips Respironics for assistance.
- Repairs and adjustments must be performed by Philips Respironics-authorized service personnel only. Unauthorized service could cause injury, invalidate the warranty, or result in costly damage.
- To reduce the risk of bodily injury and damage to the equipment, retain and follow all product safety and operating instructions.
- The Health Industry Manufacturers Association recommends that a minimum separation of six inches be maintained between a wireless phone and a pacemaker to avoid potential interference with the pacemaker. The DreamStation Cellular Modem should be considered a wireless phone in this regard.
- If you use any other personal critical medical device, such as pacemaker, consult the manufacturer of your device to determine if it can co-exist in close proximity with our device in terms of RF interferences. Your physician may be able to assist you in obtaining this information
- Do not use this product in gas stations, fuel stores, chemical plants and locations containing explosives.
- Do not use accessories, detachable parts, and materials not described in the instructions for use.

Cautions

- Remove power from the therapy device by unplugging the power supply cord's connector from the power inlet on the back of the device before proceeding with the installation instructions.
- If the modem has been exposed to either very hot or very cold temperatures, allow it to adjust to room temperature before beginning the installation procedure.
- Pins of connectors should not be touched. Connections should not be made to these connectors unless ESD precautionary procedures are used. Precautionary procedures include methods to prevent build-up of electrostatic charge (e.g., air conditioning, humidification, conductive floor coverings, non-synthetic clothing), discharging one's body to the frame of the equipment or system or to earth or a large metal object, and bonding oneself by means of a wrist strap to the equipment or system or to earth.


Important! Refer to the therapy device manual for additional warnings and cautions.

Important! From time to time, electronic devices can be affected by irregularities in the electrical power supplies and other electrical equipment. To prevent electrical interference that could affect the performance of your medical

equipment, always keep accessories (power supplies, battery packs, etc.) and other electrical devices (telephones, radios, etc.) away from your CPAP when used with the cellular modem.

Symbols


Symbol	Definition
	Consult accompanying instructions
	This device contains an RF transmitter
IP22	Drip proof equipment
	Separate collection for electrical and electronic equipment per EC Directive 2012/19/EU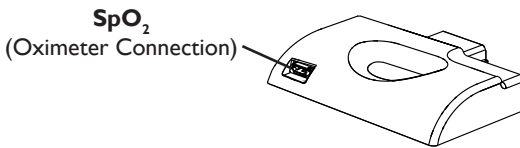

Installing the Modem


In order for the modem to work properly, it must be installed in a therapy device.

1. Ensure the therapy device is turned off.
2. Open the accessory door on the therapy device.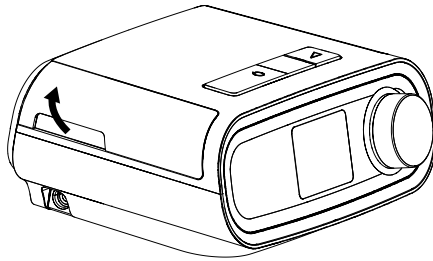


Open Accessory Door

3. Slide the modem into the accessory slot until it clicks into place.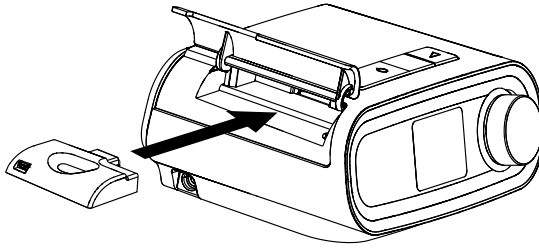

Slide Modem into Place

4. Close the accessory door on the therapy device.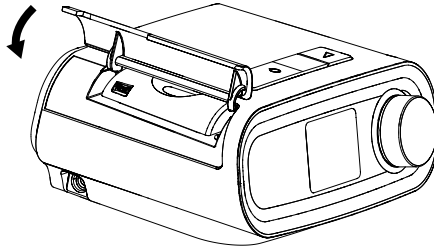

Close Accessory Door

Note: After powering on, up to 30 seconds may elapse before the modem is ready to make a call.

Important! The modem will automatically call within 5 minutes of installation if the blower is off and a manual call has not been successfully completed.


Making a Call

Normally, the modem will automatically make a call and upload therapy data at predetermined times set by your provider. The modem will not make a call while the blower is on and therapy device is delivering therapy.


If instructed to do so by your provider, you can manually make a call by navigating to the My Provider menu on the therapy device and selecting "Upload." The device will then display the animated Sending screen, then display that the upload was completed.

My Provider Screen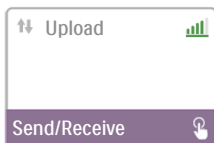

Upload Screen

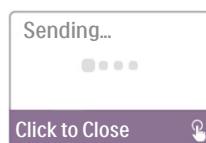Animated Sending Screen

Upload Complete Screen

Note: When located near a clock radio, speaker, or other electronic device, an occasional buzzing noise may be heard. Relocate the therapy device and modem away from the electronic device to reduce or eliminate this noise.

Modem On/Off


If necessary, you may turn the modem off temporarily by navigating to the My Setup menu and selecting “Modem Off.” The modem will not be able to make any calls while off, and an airplane icon will appear on the upload screen. When you are ready to turn the modem back on, navigate to the My Setup menu and select “Modem On.”

My Setup Screen

Modem Off Icon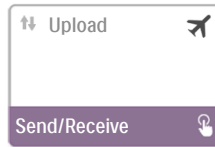


Upload Screen when Modem is Off

Modem On Icon

Error Messages

If a connection or upload attempt fails, an error message will be displayed on the therapy device screen indicating the error number.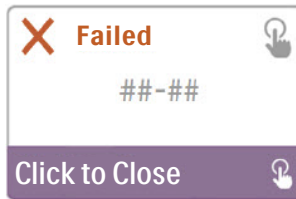

Example of an Error Message


Number	Displayed Text	Meaning	Action
1-0 1-1	Carrier Network Issue	Connection Error	Try removing power from the therapy device by unplugging the power supply cord's connector from the power inlet on the side of the device, wait 10 seconds, then plug the cord in again. The, try initiating a manual call by navigating to the My Provider menu on the therapy device and selecting “Upload.” See the “Making a Call” section of this guide. If problem persists, contact your provider. The provider should check the Philips Respironics proprietary compliance software for additional information.
1-2 1-3 1-4	Call Again Later		
2-0 2-1 2-2 2-3 2-4 2-5	Call Again Later		

Number	Displayed Text	Meaning	Action
3-0	Call Product Support	Software Configuration Error	Try initiating a manual call by navigating to the My Provider menu on the therapy device and selecting "Upload." See the "Making a Call" section of this guide. If problem persists, contact your provider. The provider should check the Philips Respironics proprietary compliance software for additional information.
3-1	Check Enore Setup		
3-2	Call Again Later		
4-0 4-1 4-2 4-3	Modem Service Required	Invalid Device Configuration Error	Contact your provider. The provider should check the Philips Respironics proprietary compliance software for additional information.

Note: Philips Respironics proprietary compliance software is developed by Philips Respironics and is intended for use by physicians and home care providers.


Removing the Modem

1. Ensure the therapy device is turned off.
2. Open the accessory door on the therapy device.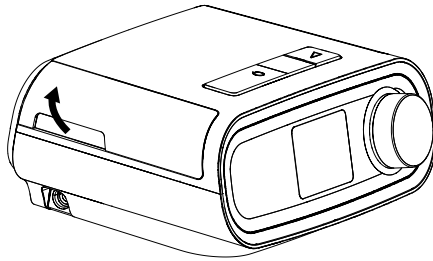

Open Accessory Door

3. Gently, pull the modem out of the therapy device.

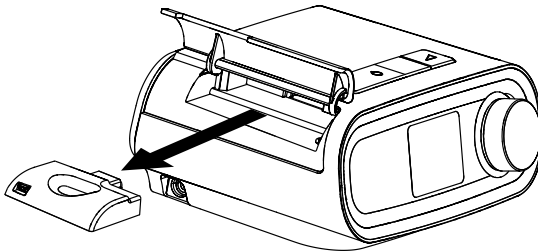Remove the Modem

4. Close the accessory door on the therapy device.

Temperature and Storage Information

	Operation	Transport & Storage
Temperature:	41° to 95° F (5° to 35° C)	-4° to 140° F (-20° to 60° C)
Humidity:	15% to 95% non-condensing	15% to 95% non-condensing

Disposal

Separate collection for electrical and electronic equipment per EC Directive 2012/19/EU. Dispose of this device in accordance with local regulations.

Additional Notices

- Notice:** The antenna(s) used for this transmitter must be installed to provide a separation distance of at least 30 cm from all persons and must not be co-located or operating in conjunction with any other antenna or transmitter.
- Notice:** For regulatory identification purposes, the DreamStation Cellular Modem is assigned a model number of 200650C or 100650C.
- Notice:** There are no user-serviceable parts within the cellular modem accessory.
- Notice:** The cellular modem is not suitable for use on airlines.
- Notice:** Use of non-original manufacturer-approved accessories may violate your local RF exposure guidelines and should be avoided.

Hereby, Respiration Inc. declares that this class 1 radio equipment is in compliance with Directive 2014/53/EU. The full text of the EU declaration of conformity is available at the following internet address: <http://incenter.medical.philips.com/PMSPublic>

Product Support

Should you experience trouble with this equipment or require assistance setting up, using, or maintaining the equipment, contact your home care provider or Philips Respiration at 1-800-345-6443 or 1-724-387-4000.

Specifications

- The cellular modem has an expected service life of 5 years.
- Not all bands are supported by local carriers.
- Mode of Operation: Continuous

Radio Specifications

Band DCS1800:

Tx Uplink: 1710-1785 MHz
Rx Downlink: 1805-1880 MHz
Channel Bandwidth: 200 KHz
Modulation: GMSK
Output power: +30dBm +/- 2dB

Band PCS1900:

Tx Uplink: 1850.2 ~ 1909.8 MHz
Rx Downlink: 1930.2 ~ 1989.8 MHz
Channel Bandwidth: 200 KHz
Modulation: GMSK
Output power: +30dBm +/- 2dB
(+24 dBm +1/-3 dBm)

Band IMT UMTS 2100,WCDMA FDD:

Tx Uplink: 1922.4-1977.6 MHz
Rx Downlink: 2112-2167 MHz
Channel Bandwidth: 5 MHz
Modulation: QPSK/DQPSK
Output Power: +24 dBm +1/-3 dBm

Band CLR UMTS 850,WCDMA/FDD:

Tx Uplink: 826.4 ~ 846.6 MHz
Rx Downlink: 871.4 ~ 891.6 MHz
Channel Bandwidth: 5 MHz
Modulation: QPSK/DQPSK
Output Power: +24 dBm +1/-3 dBm

Limited Warranty

Respironics, Inc., a Philips company ("Philips Respironics") provides this non-transferable, limited warranty for DreamStation Cellular Modem ("Product") to the customer who originally purchased the Product directly from Philips Respironics.

What this Warranty Covers: Philips Respironics warrants each new Product will be free from defects in materials and workmanship and will perform in accordance with the Product specifications under normal and proper use and maintenance in accordance with applicable instructions, subject to the exclusions below.

How Long does this Warranty Last: Two (2) years from the longer of the date of shipment to the purchaser or date of setup by purchaser for the end user.

What this Warranty does not cover: This warranty does not apply to any software included with the Product as the software warranty is included in the software license. This warranty does not cover damage or injury whether to the Products, personal property, or persons caused by accident, misuse, abuse, Acts of God, water ingress, repair or alteration by anyone other than Philips Respironics or its authorized service center, failure to operate in accordance with the terms of the operating manual and instructions, lack of reasonable care, the discontinuance of a network (e.g. 2G, 3G, etc.) by a carrier (e.g. ATT, Verizon, etc.), or other defects not related to material or workmanship. This warranty is not transferable. If Philips Respironics finds that a Product returned for service or the issue raised is not covered under this limited warranty, Philips Respironics may charge an evaluation fee and return shipping.

What Philips Respironics will do: If a Product does not meet the warranty above in the first 90 days after the original shipment date, Philips Respironics will replace the device with a new Product. Thereafter, if a Product fails to conform to the warranties set forth above during the applicable warranty period, Philips Respironics will repair or replace the Product or refund the original purchase price, in Philips Respironics sole discretion. Philips Respironics may use new or remanufactured assemblies, components, and parts in repair and new or recertified refurbished devices for replacement. The balance of the original warranty period will apply to any Product or component of a Product repaired or replaced under this warranty.

Warranty Disclaimer; Limitation of Liability: EXCEPT AS SET FORTH IN THIS LIMITED WARRANTY, PHILIPS RESPIRONICS MAKES NO WARRANTIES, EXPRESSED OR IMPLIED, STATUTORY OR OTHERWISE, REGARDING THE PRODUCT OR ITS QUALITY OR PERFORMANCE. PHILIPS RESPIRONICS SPECIFICALLY DISCLAIMS THE IMPLIED WARRANTY OF MERCHANTABILITY AND THE IMPLIED WARRANTY OF FITNESS FOR A PARTICULAR PURPOSE. IN NO EVENT WILL PHILIPS RESPIRONICS MAXIMUM LIABILITY UNDER THESE WARRANTIES EXCEED THE ORIGINAL PURCHASE PRICE OR WILL PHILIPS RESPIRONICS BE LIABLE FOR ANY ECONOMIC LOSS, LOSS OF PROFITS, OVERHEAD, OR SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. Repair, replacement, or return of purchase price by Philips Respironics is the original purchaser's sole and exclusive remedy under this warranty.

This warranty gives you specific legal rights, and you may also have other rights that vary from state to state. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above exclusion and limitations may not apply to you.

How to get warranty support: Patients contact your local authorized Philips Respironics dealer and dealers contact Respironics, Inc. at:

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